


Rev. 00 // 07/11/2013


1.)

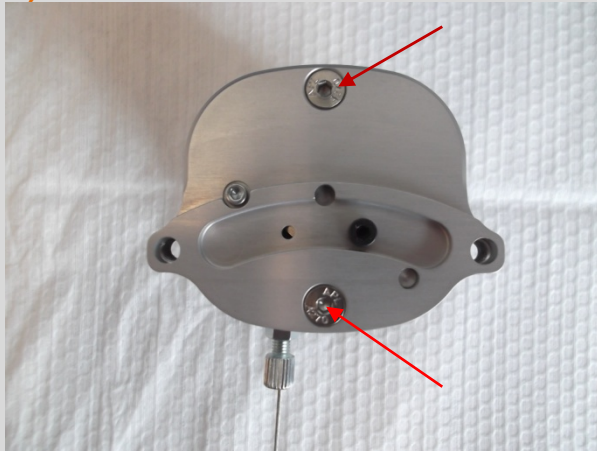

Move into 1<sup>st</sup> gear

Loosen countersunk head screws and carefully remove casing lid.


2.)

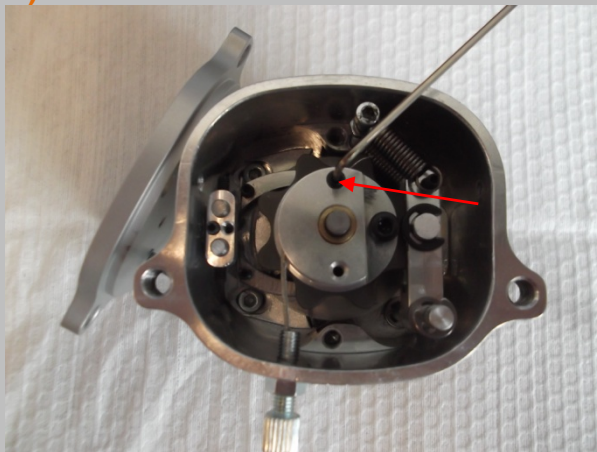

Loosen the cable retaining screw and remove the defective cable.

Search casing for broken pieces of cable and also remove these.


3.)

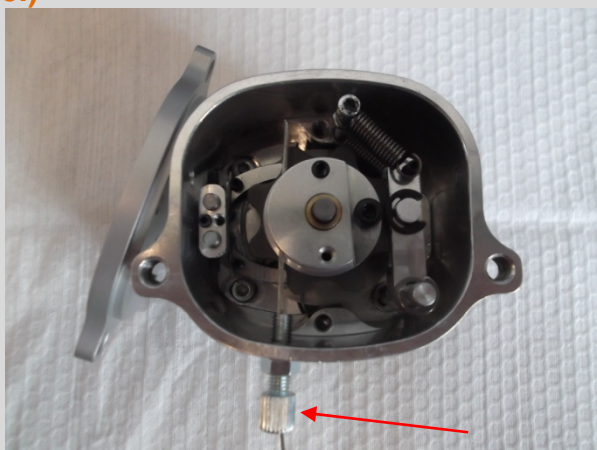

Thread in new cable.

Rev. 00 // 07/11/2013


4.)

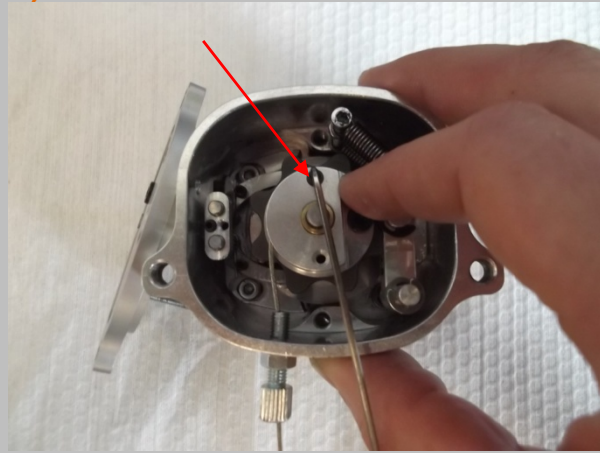

Use a finger to insert the cable along contour and tighten clamp screw.


The cable must rest tightly on the circumferential groove.

5.)

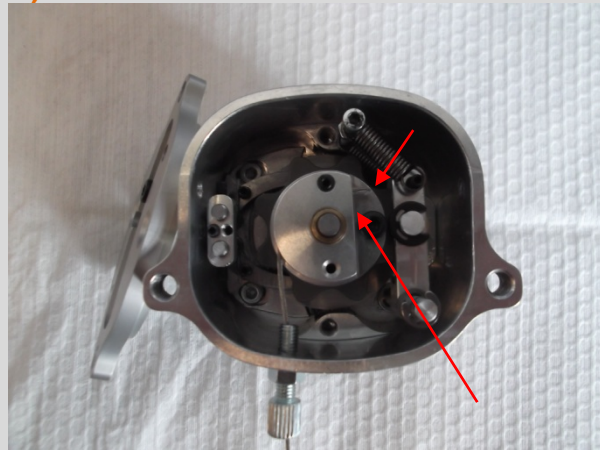

The end of the cable must protrude around 2-4 mm beyond the marked edge.

6.)

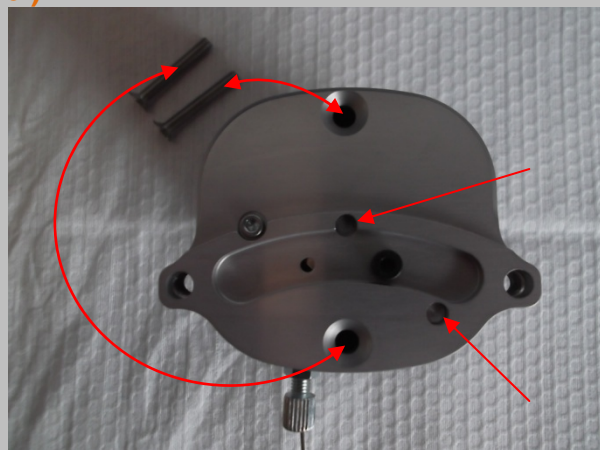

Carefully replace casing lid.

Ensure that both bolts are cleanly inserted into the lid.

The casing lid must close without gaps.

Tighten the countersunk head screws evenly.