

Subsequent inspection: After another 1,000 km or 1 year after the last inspection or after heavy use

Serial number: SN _____ Kilometre reading: _____	OK / carried out	not OK	resolved
Check that all screws/fastening elements are firmly seated and replace, if necessary (particularly the following: M8x35 oval head screws for the handle rotation axes on the cranks), check and adjust the eccentric clamp lever on the bottom bracket support	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Clean and oil/grease all pivot points and bearings	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Carry out a visual inspection of the frame and attachments for crack formations, deformations, etc.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Carry out a functional and safety check of the brakes and, where necessary, replace the brake fluid, brake pads, brake cables	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Check, adjust, clean and oil the gear components, check the chain deflector guide on the luggage rack (if present), check the chain tension, check the chain sliding blocks for wear	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Check the electrical connections and the performance of the rechargeable batteries	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Check of the control parameters and functionality of the drive system; perform a software update if necessary	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Check the spoke tension of the drive wheel and, if required, correct the tension/recentering and inspect the axle nuts of the drive wheel for firm seating (tightening torque for Shimano: 40 Nm)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Carry out a functional and safety check of the drive wheel and, if required, replace the tyre on the product	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Carry out a functional and safety check of all lights (if fitted), steering and adaptation on the product and the adapted wheelchair/wheelchairs	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Carry out a functional and safety check of the adapter	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Test drive/functional test	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

OK / carried out = OK | not OK = not OK | resolved = the fault was corrected

Comments:

Rehabilitation specialist dealer:

First name and last name of contact:

Stamp:

Date/signature

To maintain the warranty, the completed inspection list must be sent by e-mail or post to PRO ACTIV within four weeks of the inspection.